



Distribuito in ITALIA da  
**Li StarFish S.r.l.**  
Via Cavour, 35  
20063 Cernusco S/N (MI)  
telefono 02-92150794  
fax 02-92157285  
info@listarfish.it  
www.listarfish.it

# DIPFIT®

## **CORONAVIRUS - BIO K 385** (BOX OF 10 TESTS)

**TEST STRIPS FOR  
DETECTION OF  
CORONAVIRUS  
IN CALF FAECES**



## **HOW TO PROCEED**

**1**



Take the faeces directly from the rectum of the calf. If the samples are liquid, take a spoonful.

**2**



If the samples are solid, remove the excess amount with a spatula or clean object.



Dilute the sample with the liquid contained in the dilution tube. Homogenize well, taking care to prevent foam formation.



**4**


Tear the aluminium envelope open at the notch. Once the device has been taken out of the envelope its stability is of short duration, especially in a humid environment.



**5**

Plunge a device into the liquid.

Wait at most 10 minutes and interpret the result by comparing it with Photo 6.

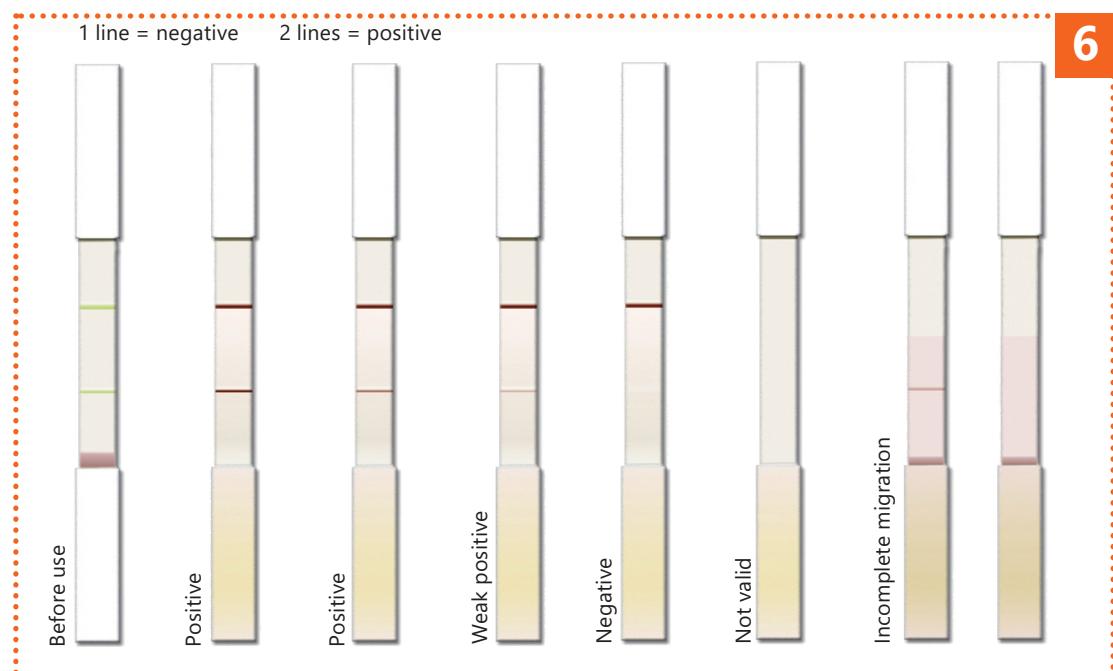

#### Precautions for use

- Store kits in a dry place at room temperature.
- Wear gloves while carrying out the test.
- The analysed sample shouldn't be too concentrated. Do not prepare a sample volume greater than a spoonful.
- Dispose of the device in keeping with legislation on environment protection and management of biological waste.

